


EL

ESTHETIC LINE implant

C-TECH INSTRUMENTS NECESSARY FOR EL IMPLANT SURGERY

DENTAL IMPLANT

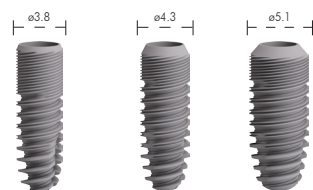

7 mm		EL-4307	EL-5107
9 mm	EL-3509	EL-4309	EL-5109
11 mm	EL-3511	EL-4311	EL-5111
13 mm	EL-3513	EL-4313	EL-5113
15 mm	EL-3515	EL-4315	EL-5115



Locator drill
CT-2020



2.0 external irrigation drill (Ø 2.0)
CT-1720E



3.5 external irrigation drill (Ø 3.0)
CT-1735E



4.3 external irrigation drill (Ø 3.6)
CT-1743E



5.1 external irrigation drill (Ø 4.6)
CT-1751E



Counterbore
CT-2035E



Hard bone drill
3.5 mm
EL-1735N



Hard bone drill
4.3 mm
EL-1743N



Hard bone drill
5.1 mm
EL-1751N



Drill Extender
CT-2000



Manual Hex Driver
CT-9025S



1.25mm Hextool Torque Wrench
Attachments CT-8051



1.25mm Hextool Torque Wrench
Attachments (Long) CT-8052



Implant Latch Driver
CT-E9040



Implant Ratchet Driver Short
BL-E7001



Implant Ratchet Driver Long
BL-E7001L



Depth Gauge (3.5/ 4.3/ 5.1)
CT-E9007
CT-E9008
CT-E9010




Paralleling Pins, qty. 2
(1.6 mm & 2.0 mm) CT-9000



Set metal stopper (L.9/11/13/15)
CT-Stop 07/08/09/10



Abutment Extractor Screw
BL-6060

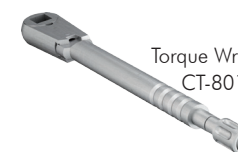

Torque Wrench (50 Ncm)
CT-8010

C-TECH
IMPLANT

www.c-tech-implant.com



EL ESTHETIC LINE implant SURGICAL PROCEDURE


C-TECH
IMPLANT

SURGICAL PROCEDURE BL Ø 3.5 IMPLANT

D2/D3

D1 ADDITIONAL STEPS

Explanation of drill marking

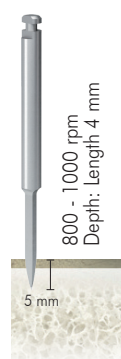

CT-2020
Locator Drill



CT-1720E
2.0mm Drill
(Outer Ø 2.0mm)




CT-1735E
3.5mm Drill
(Outer Ø 3.1mm)




BL-E7001L
Implant
ratchet driver



EL-1735N
3.5 mm
Hard bone drill

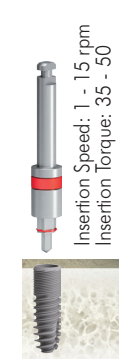


BL-E9040
Implant latch

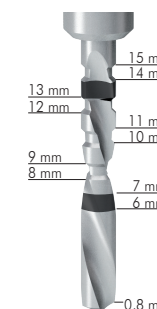

SURGICAL PROCEDURE BL Ø 4.3 IMPLANT

D2/D3

D1 ADDITIONAL STEPS

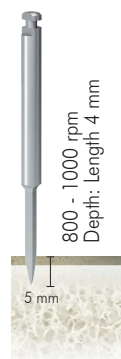

CT-2020
Locator Drill



CT-1720E
2.0mm Drill
(Outer Ø 2.0mm)



CT-1735E
3.5mm Drill
(Outer Ø 3.1mm)




CT-1743E
4.3mm Irr. Drill
(Outer Ø 3.9mm)



BL-E7001L
Implant
ratchet driver



EL-1743N
4.3 mm
hard bone drill

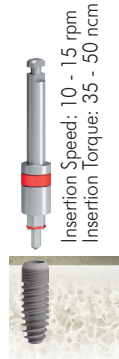


BL-E9040
Implant latch

SURGICAL PROCEDURE BL Ø 5.1 IMPLANT

D2/D3


D1 ADDITIONAL STEPS

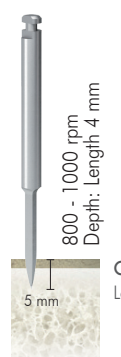

CT-2020
Locator Drill



CT-1720E
2.0mm Drill
(Outer Ø 2.0mm)

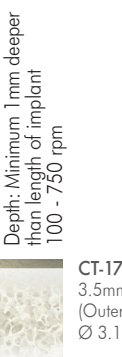

CT-1735E
3.5mm Drill
(Outer Ø 3.1mm)




CT-1743E
4.3mm Irr. Drill
(Outer Ø 3.9mm)




CT-1751E
5.1mm Drill
(Outer Ø 4.6mm)



BL-E7001L
Implant
ratchet
driver

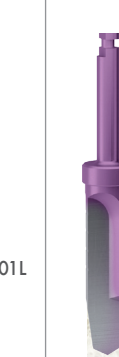

EL-1751N
5.1 mm
hard bone drill

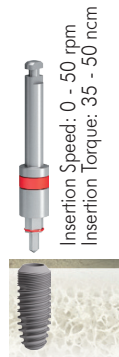

BL-E9040
Implant latch

IMPORTANT: An additional 0.8 mm must be added to the length of the drill to account for the angled cutting tip.
* Depth: Minimum 1mm deeper than length of implant is to allow for subcrestal seating.

NOTE: Metal stoppers can not be mounted on Ø 5.1 drills

